Wir hören uns.

**AUDIA**

# AUDIA ELACIN

RC19 | RC18 | RC17 | RC15 | RC13\*

**EN** | User Manual  
**NL** | Gebruiksaanwijzing  
**DE** | Gebrauchsanleitung  
**FR** | Mode d'emploi  
**ES** | Manual del usuario  
**IT** | Manuale dell'utente  
**DK** | Brugervejledning


Custom-made hearing protection  
Op maat gemaakte gehoorbescherming  
Maßgefertigter Gehörschutz  
Protection auditive sur mesure  
Protetores auditivos personalizados  
Otoprotettori personalizzati  
Specialfremstillede ørepropper


1. **EN** | User Manual
2. **NL** | Gebruiksaanwijzing
3. **DE** | Gebrauchsanleitung
4. **FR** | Mode d'emploi
5. **ES** | Manual del usuario
6. **IT** | Manuale dell'utente
7. **DK** | Brugervejledning

## INSERTION


NL | Inbrengen - DE | Einsetzen - FR | Mise en place de la protection - ES | Colocación -  
IT | Inserimento - DK | Isætning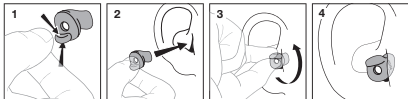

**AUDIA ELACIN**  
**RC19 | RC18 | RC17 |**  
**RC15 | RC13\***

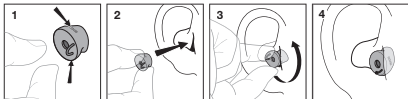**AUDIA ELACIN**  
**RC19 | RC18 | RC17 |**  
**RC15 | RC13\* MINIGRIP**

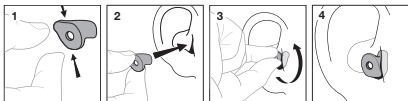**AUDIA ELACIN**  
**RC19 | RC18 | RC17 |**  
**RC15 | RC13\* NO GRIP**

## ATTENUATION CHARACTERISTICS

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<b>EN</b>	<b>f in Hz</b>	Frequency in Hertz.
	<b>H in dB</b>	Attenuation value in predominantly high-frequency noise spectrum ( $f > 2000$ Hz).
	<b>M in dB</b>	Attenuation value in predominantly medium-frequency noise spectrum ( $f > 500$ and $f < 2000$ Hz).
	<b>L in dB</b>	Attenuation value in predominantly low-frequency noise spectrum ( $f < 500$ Hz).
	<b>SNR in dB</b>	Single Number Rating, to be deducted from C-weighted noise level.
	<b>Mf in dB</b>	Average attenuation value at given frequency $f$ in decibel.
	<b>Sf in dB</b>	Standard deviation at given frequency $f$ in decibel.
	<b>APVf in dB</b>	Assumed Protection Value ( $Mf - 1x Sf$ ), at given frequency $f$ in decibel.

Specifications: The European type certification inspection was performed according: EN 352-2: 2002 by IFA, 53757 Sankt Augustin, Germany, Kenn.no.: 0121.

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**EN** The RC13 filters do not fulfil the minimum attenuation requirements according to the EN 352-2. For daily noise exposure levels  $L_{ex,8h}$  of 88 dB(A) (for the RC13), a residual sound level at the ear of  $L_{ex,8h} = 79$  dB(A) is reached. If the exposure time is halved to 4 h, a level at the ear of  $L_{ex,8h} = 79$  dB(A) is also reached for a noise exposure level of  $L_{Aeq} = 90$  dB(A).

**The hearing protectors are suitable for 8 h use per day in following areas:**

- Swimming pool (except in the machine rooms)
- Fitness Gym room
- Music school teachers except: percussion and brass / wind instruments,
- Individual acoustical musicians, except brass / wind instruments, singers and percussion,
- Pubs
- Restaurants (without loud music)
- Gardening with power tools up till 400 h/yr,
- Call centers
- Aircraft passenger cabin
- Garment Industry
- Laundry Service & Dry Cleaning
- Harbour Outdoor worker
- Forklift driver (without loud background noise)
- Printing Houses except energy supply dept.
- Plastic & Leather Industry.

The typical noise situations in these areas, the daily noise exposure level at the ear  $L_{ex,8h}$  does not exceed the value of 85 dB(A). Thus a secure protection is generally assured by the RC13. For particular cases with unusual loud situations the RC13 filters may not be suitable. In case of any doubt, please contact your supplier or [info@audia-akustik.de](mailto:info@audia-akustik.de) for assistance or further information.

The attenuation of the RC13 do not fulfill the EN352 requirement, but do satisfy all relevant requirements of the PPE directive (EU) 2016/425. The RC13 are certified by IFA (Reg.No. 0121), 53757 Sankt Augustin, Germany.

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## AUDIA ELACIN RC CUSTOM-MADE HEARING PROTECTION

**IMPORTANT:** Read this instruction carefully before use and keep both the package and the user manual.

### General

The custom-made hearing protection ear mould AUDIA Elacin RC protects your hearing from harmful noise emissions whilst being comfortable to wear. It is intended for a service life of at least two years. The Elacin RC13\* filters are designed to prevent overprotection in relatively low noise level areas and to maintain optimal speech intelligibility and perception of warning-signals/-messages and work related sounds.

### Use

Check that the hearing protectors are clean before you use them. If necessary use an ear wax remover<sup>†</sup> to clean ear wax out of the sound tube of the hearing protector. Note the red and blue color marks on your hearing protectors: red = protector for the right ear; blue = protector for the left ear. Your hearing specialist will be glad to demonstrate the correct way to insert and remove the hearing protectors.

<sup>†</sup> An ear wax remover is available separately. Ask your hearing specialist for more information.

**Insertion**

Insertion may be less smooth at first use. Application of ear cream on the part of the hearing protector that goes into the ear will improve ease of insertion. See picture on page 4.

**Removal**

Remove the hearing protector using a rotary forward motion. OR, alternatively, place your index finger against the lower part of your ear and press upwards a number of times. The earpiece is then slowly released from the ear. When it no longer seals you can safely remove it.

**Maintenance**

Clean your AUDIA Elacin RC hearing protectors regularly with a damp cloth. For a regular and thorough cleaning of your custom-made hearing protectors, we advise you to use the cleaning products of the manufacturer. These cleaning products help you to keep your hearing protectors in the best possible shape and prolong the life-span. For further information, please contact your hearing protection representative or your hearing care professional. Store the ear protection device in the packaging provided when not in use.

## **Warning**

- Use AUDIA Elacin RC hearing protectors carefully in accordance with the instructions. If the instructions above are not followed, the protection certainty of the hearing protectors cannot be guaranteed.
- Only enter the noise area with your individually customised ear protection device if you are familiar with how to use the device and correct fit in the auditory canal has been ensured. Ensure that you have been shown how to insert it correctly, e.g. by the manufacturer or supplier of the hearing protection ear mould or your company doctor. At the beginning, it can be easier to practice this using a mirror.
- Use your AUDIA Elacin RC hearing protectors always in noisy situations! NOTE: the protection is reduced substantially even if you only remove them for a very short period. You will not be adequately protected if you fail to wear the hearing protectors at all times.
- This product may be adversely affected by certain chemical substances. Further information should be sought from the manufacturer.
- If applicable: These hearing protectors should not be used where there is a risk that the connecting cord could be caught up during use, for example, by a machine.
- The hearing protection device must not be pulled out of the ear canal by the optional cord, as this may damage your hearing protection device. To remove the device only hold on to the protruding material at the ear canal entrance or the optional grip.
- Sudden or fast removal of the hearing protectors out of the ear canal may damage the ear drum.

- The hearing protectors are custom-made and are therefore not transferable. The hearing protectors are designed to be inserted in the hearing canal only.
- We recommend that you replace your hearing protection device after four years at the latest.
- The greatest possible care is taken in the manufacturing of your AUDIA Elacin RC hearing protectors. Immediately contact your supplier if you notice anything out of the ordinary.
- Anatomical changes in the ear shape can negatively affect the seal of the ear moulds, e.g. in cases of weight change or ear surgery. Safe protection is only ensured if the hearing protection device ear mould is checked after delivery and thereafter at regular intervals of a maximum of three years.
- Please bear in mind that, when hearing protection is used, you will also hear signals in your environment more quietly (e.g. warning signals and voice signals). In addition, the regulations on the use of hearing protection in road traffic/track superstructure must be observed.
- The hearing protection device is made exclusively from biocompatible materials that have been proven and tested over a number of years. However, if you do notice any issues, remove the hearing protection device immediately and inform your doctor, specialist advisor or supervisor.
- Choking hazard for infants! The product, its components and packaging must be kept out of the reach of children.
- The product may not be modified or changed.

### **Declaration of conformity**

AUDIA AKUSTIK GMBH hereby declares that the hearing protection ear mould AUDIA Elacin RC complies with PSA Regulation (EU) 2016/425.

The complete declaration of conformity can be viewed on our website at: [www.audia-akustik.de](http://www.audia-akustik.de)  
(Download area)

Relevant notified body for conformity assessment:

PZT GmbH

Bismarckstraße 264 B

26389 Wilhelmshaven | Germany

Identification number: 1974

You can obtain further technical information from the following address:

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Elacin is a registered trademark
**ELACIN INTERNATIONAL B.V.**
of Elacin International B.V.